

D20

ANESTHETIC VAPORIZER



- 5 Year Warranty
- Calibrated for Life*
- The Vetland D20 vaporizer is available with funnel or key fill systems for Isoflurane and Sevoflurane
- Can be universally mounted to a variety of anesthesia systems
- Standard with 23 mm Cagemount Manifold†

* Calibrated for Life: Scheduled rebuilds are not required to tune the D20 Vaporizer. Efficacy output tests are suggested as a part of your annual anesthesia machine maintenance.

5 Year Warranty - Calibrated for Life

The Vetland D20 anesthesia agent vaporizer is automatically temperature compensated. The temperature compensation mechanism is resistant to aging and allows Vetland Medical to offer a 5-year warranty and promote this as a “calibrated for life” vaporizer because the vaporizer has no set recalibration or adjustment schedule. The vaporizer does not have an expiration on the calibration and does not require it to be drained, removed, and sent in for rebuild and adjustment in a clean lab as other competitive vaporizers require.

Vetland Medical does recommend, as a part of your normal routine machine maintenance, that the efficacy output test of the D20 be performed to verify proper function but, again, unless this test finds the performance outside of specifications, there is no need to send the D20 vaporizer in for recalibration on any set schedule.

The Vetland D20 can be universally mounted to a variety of anesthesia systems. The D20 provides up to 5% Isoflurane and 8% Sevoflurane with flow volumes from 300cc to 10LPM.

Vetland Medical Sales & Services, LLC

2601 Holloway Road Louisville, KY 40299 | Phone: 1-502-671-1014 | Fax: 1-502-671-1019
www.vetlandmedical.com | 1-866-476-0589 US Toll Free

Vetland D20 Vaporizer – With 5 Year Warranty

| | |
|--|--|
| Condition | New |
| Warranty | 5 Years |
| Agents Available | Isoflurane or Sevoflurane |
| Fill Style | Funnel Fill or Key Fill |
| Mounting | Cage Mount (†custom option: Drager or Selectatec) |
| Accuracy | ± 15% |
| Agent Capacity | 150 ml |
| Concentration Range (volume at flow rates from 300 cc to 10 LPM oxygen) | Iso: 1-5% Sevo: 1-8% |
| Weight | 4 kg / 8.818 lbs |
| Dimensions | 185 mm x 140 mm x 135 mm |
| Flow | 1 L to 10 LPM |
| Temp Range | 59° F – 90°F |



Vetland Medical's Vaporizer Exchange Program

Many vaporizers have manufacturer recommendations that they are cleaned and calibrated annually to ensure accuracy. Previous options for this function were down time with your equipment while you sent your vaporizer in for service, or an incomplete overhaul if you had a technician perform the service in house. Now, the process is easy with Vetland Medical's Vaporizer Exchange Program!

How It Works:

1. Call us to place an order for a vaporizer exchange (please specify vaporizer type, agent, model and brand of machine if it is to be mounted to)
2. When replacement vaporizer arrives:
 - Empty the agent from and detach your existing vaporizer from your anesthesia machine
 - Attach the new, refurbished vaporizer to your anesthesia machine
 - Place your old unit in the shipping box, apply the pre-paid return-shipping label, and send the unit to us

Vetland Medical Sales & Services, LLC